




THE REVOLUTIONARY AMA RUT Pro



DESCRIPTION OF THE METHOD

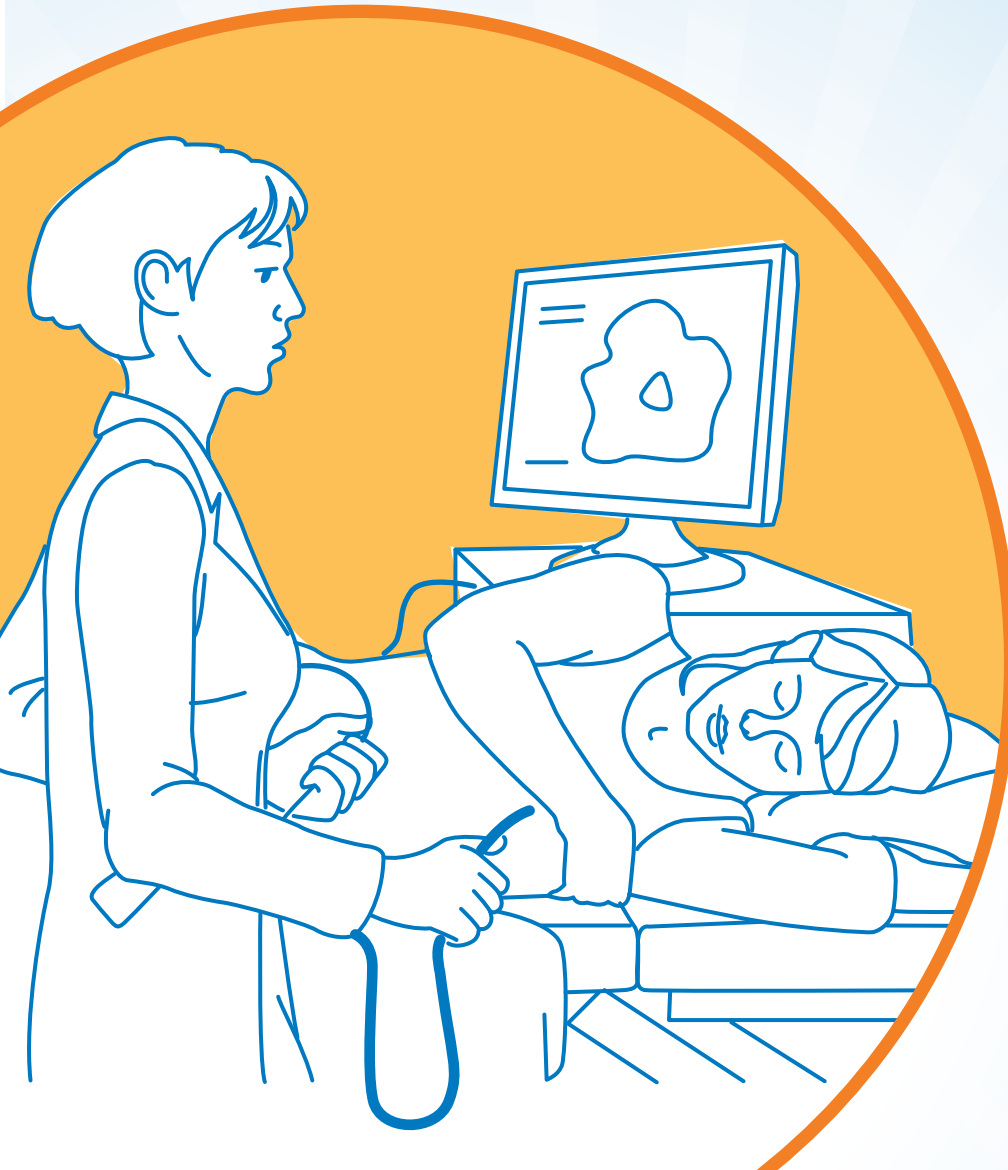


- ✓ Invasive biochemical method for *H.pylori* detection;
- ✓ Endoscopy procedure;
- ✓ Determines a high urease activity in the biopsy specimen;



DESCRIPTION OF THE METHOD

Based on the color change of the indicator in the medium alkalization with ammonia, formed by hydrolysis of urea with *H.pylori* urease;

RUT
perfect

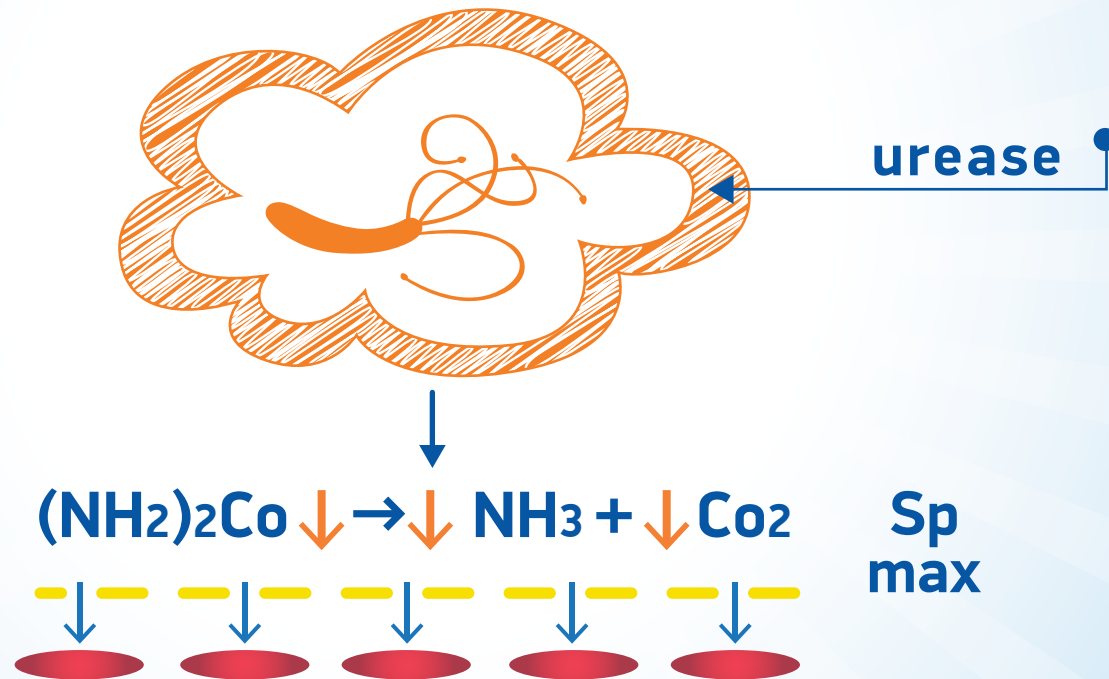

AMA RUT IS A DRY COOL RAPID UREASE TEST



DIAGNOSTICS BECOMES SMART

The highest level of accuracy due to innovative structure:
99% sensitivity,
99% specificity.
Reliable guarantee of the target reactions only.





AMA RUT Pro

Test gives result within 5 min. The absence of pink spot after 5 min means the negative result.



Unique materials
allowing work in the
broad temperature
range and storage
without special
conditions for
24 months.



shelf-life 24 months



AMA RUT Pro

Slim packaging
in the light flat
envelope




ECONOMY
DESIGN



Budget-friendly price

Open one
test, use
other later


Twenty four test-
slides in the pack

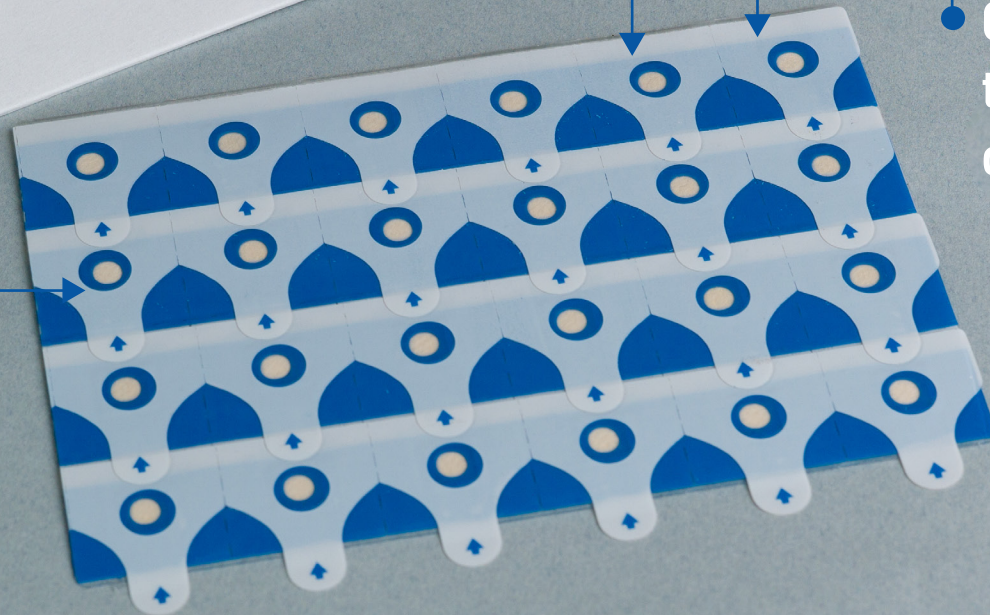



NEW DESIGN ALLOWING TO CHOOSE ANY AMOUNT OF BIOPSIES TAKEN FROM ONE PATIENT


- Decision about the needed amount of biopsy samples to be taken right in the course of the Endoscopy examination.

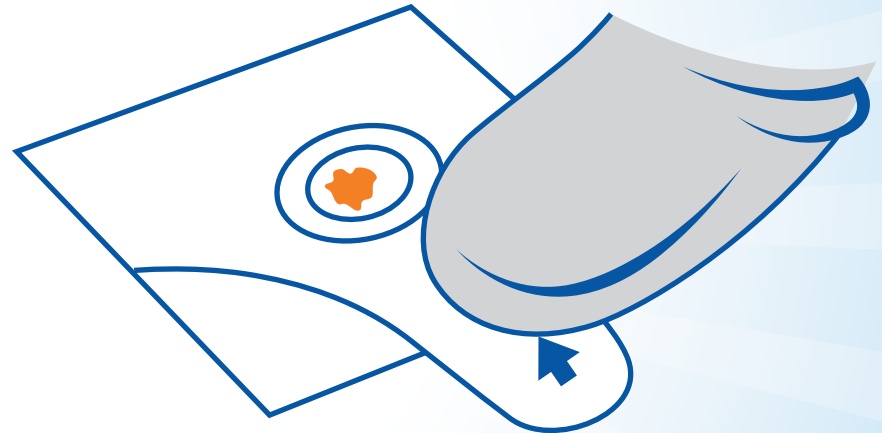


- 1, 2, 3 or 5 specimens from one or different parts of Stomach according to Guidelines and doctor's view.




TESTING PROCEDURE

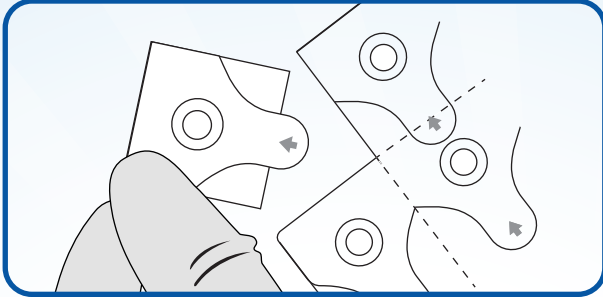

1) Separate the required number of the test segments in the line of perforation and open the protective cover gently

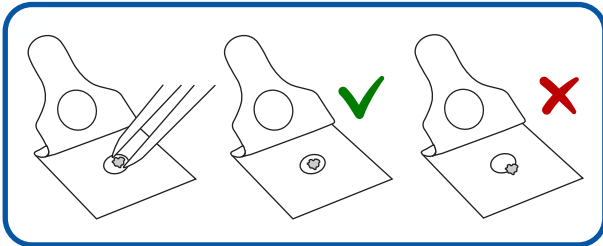

2) Place the biopsy specimen on the reactive element of the test segment.

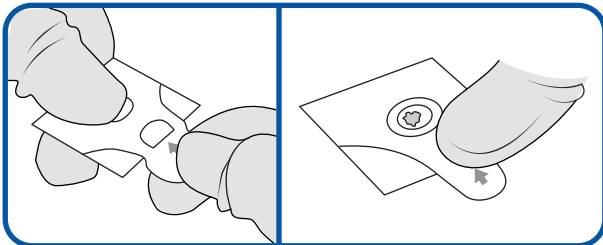

3) Reseal the cover accurately and start timing.

● ● **HP+** ● **HP-**

4) Observe the result after 5 minutes.



DIAGNOSTICS BECOMES SMART

**FAST RESULTS (UP TO 5 MINUTES)
THANKS TO SPECIAL DESIGN
OF THE REACTIVE ELEMENT**



01:59



AMA RUT Pro compare to other RUTs



The biopsy specimen needs not to be squeezed, which saves it for a further examination



AMA RUT Pro compare to other RUTs



- The test is performed in room temperature without incubation, thus the specificity is higher;
- The positive reaction comes as a response to the urease present in the tissue, not cultivated;



**THANK YOU FOR YOUR
ATTENTION!**



www.amarut.pro